

# Carson Recliner

## For Healthcare Environments

### Cleaning Instructions



#### Crypton Fabric

Clean with soft cloth dampened with water. Spots can be removed by scrubbing with a mixture of water and enzyme laundry detergent (2 cups water : 1 teaspoon detergent). Tougher stains should be treated professionally. We do not recommend cleaning crypton with any mixture containing bleach. Note that seat and back upholstery covers can be removed for better cleaning.

#### Vinyl

Clean with soft cloth dampened with warm water and mild soap solution. Stubborn stains and spills may be treated with a sponge or soft brush and mild non-abrasive cleaner. For greater accuracy please refer to the cleaning instructions from the fabric supplier, for the specific fabric in question. Note that seat and back upholstery covers can be removed for more thorough cleaning.

#### Wood Arm Caps

Wood arm caps are protected by a stain-resistant pre-catalyzed finish. For normal cleaning, wipe the surface with a soft, damp (not wet) cloth, with a mild detergent solution. We recommend full-strength or diluted glass cleaner products since they do not cause surface smearing or haze. A few examples of alternative cleaners include: Bleach, Hi-Tor<sup>®</sup> Plus, Sani-Master, Ammonia, Sursan, Matar<sup>®</sup>, TOR<sup>®</sup> HB, Quat Disinfectant, Virox<sup>™</sup>, and Virex<sup>®</sup> CaviWipes<sup>™</sup>. These cleaners must be used according to the manufacturer's instructions. Avoid chemical and/or abrasive cleaning compounds. We also do not recommend using brand-name furniture polishes. While they will not damage the surface, they may cause smearing and haze. Additionally, products containing silicone or wax content should be avoided for best long-term results.

#### Urethane Arm Caps

Wipe with mild soap and water. Do not use solvents, alcohol or stain removers.



#### Metal Inner Seat Frames

Release velcro to raise seat cushion. Dust metal seat frames with a soft, clean cloth. It may be necessary to use a mild non-abrasive cleaner. Do not use abrasive Ammonia-based cleaners. Mild non-abrasive cleaners should only be used occasionally.

#### Laminate Tabletops

Dust with a soft, clean cloth. To remove fingerprints etc. scrub with a damp soapy cloth followed by a rinse with cold water. Stubborn stains can be removed with hot water and a bit of vinegar. Avoid using abrasive cleaning tools as these may scratch the laminate surface. It may be necessary to use a mild non-abrasive cleaner.

#### Casters (if applicable)

Casters should be regularly inspected and cleaned of debris that will impede their ability to roll and swivel.