




**Vista**

**Vista Seating Collection**

**Material Content**

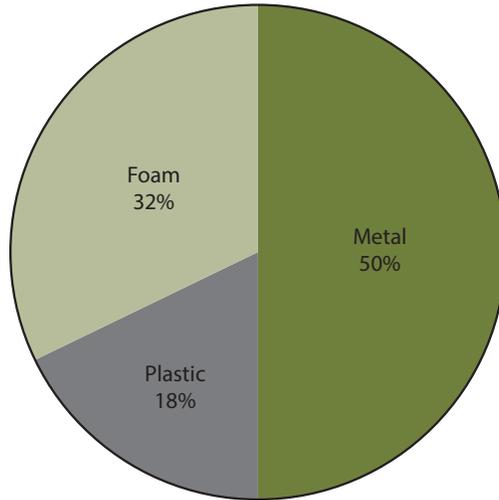

**Crypton Fabric**

Clean with a soft, damp cloth. Spots can be removed by scrubbing with a mixture of water and enzyme laundry detergent (2 cups water to 1 teaspoon detergent). Tougher stains should be treated professionally. We do not recommend cleaning crypton with any mixture containing bleach. Note that seat and back upholstery covers can be removed for more thorough cleaning.

**Vinyl**

Clean with a soft cloth dampened with warm water and a mild soap solution. Stubborn stains and spills may be treated with a sponge or soft brush and mild non-abrasive cleanser. For best results please refer to the cleaning instructions of the relevant fabric supplier. Note that seat and back upholstery covers can be removed for more thorough cleaning.

**Wood Surfaces**

Wood frames and arm caps are protected by a stain-resistant pre-catalyzed finish. For normal cleaning, wipe the surface with a soft cloth dampened with a mild detergent solution. We recommend full-strength or diluted glass cleaning products, because they do not cause surface smearing or haze. A few examples of alternative cleansers include: Bleach, Hi-Tor® Plus, Sani-Master, Ammonia,

Sursan, Matar®, TOR® HB, Quat Disinfectant, Virox™, and Virex® CaviWipes™. These cleansers must be used according to the manufacturer's instructions. Avoid chemical and/or abrasive cleaning compounds. We do not recommend using brand-name furniture polishes. While they will not damage the surface, they may cause smearing and haze. Additionally, products containing silicone or wax content should be avoided for best long-term results.

**Urethane Arm Caps**

Wipe with mild soap and water. Do not use solvents, alcohol or stain removers.

**Metal Chair Frames**

Dust with a soft, clean cloth. It may be necessary to use a mild non-abrasive cleanser. Do not use abrasive Ammonia-based cleansers. Mild non-abrasive cleansers should only be used occasionally.

**Laminate Surfaces**

Dust with a soft, clean cloth. To remove fingerprints, for example, wipe with a soft cloth dampened with warm water and a mild soap solution and rinse with cold water. Stubborn stains can be removed with hot water and a bit of vinegar. Avoid using abrasive cleaning tools such as these may scratch the laminate surface. It may be necessary to use a mild non-abrasive cleanser.

**Disclaimer:**

Numbers may vary based on model and options selected. Calculations of recycled content are based on data provided by suppliers and other available information. This data may include industry averages, other broadly based information. Stance Healthcare makes strives to provide the most accurate recycled content calculations possible but variability in market conditions or manufacturing processes may result in higher or lower content. This document will be reviewed and updated periodically and is subject to change without notice.

